



# DDS OAPro™

---

## USER GUIDE

#1



Remove plastic film off the shin guard.

#2



Always keep the direction of strap facing inward.

#3



Unstrap the top and bottom strap on one side.



# DDS OAPro™ USER GUIDE

#4



#5



#6



#7





#8



## MEASUREMENT AND SIZING

Size	Thigh 6" above the center of the knee	Center of Knee	Calf 6" below the center of the knee
S	15.5"~18.5"	13"~14"	12"~14"
M	18.5"~21"	14"~15"	14"~16"
L	21"~23.5"	15"~17"	16"~18"
XL	23.5"~26.5"	17"~19"	18"~20"

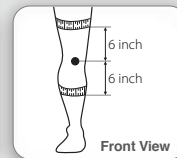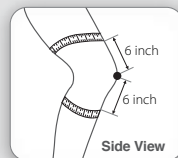

Measure the circumference of both the thigh and the calf 6" away from the center of the knee.